Exclusive Distributor

insuJet™

Needle-free
insulin injection system

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Exclusive Distributor

InsuJet™

Needle-free insulin injection system

Instructions for use

InsuJet™ Nozzle

InsuJet™
3ml Adaptor

InsuJet™
10ml Adaptor

- This packaging contains sterile consumables for use in combination with the InsuJet™ system.
- Please read the complete instructions for use booklet inside the InsuJet™ package before using the InsuJet™ system.
- Keep the original InsuJet™ instructions for use in safe custody.
- This package will only contain one type of consumable.

Warnings

- The InsuJet™ should only be used for the administration of U-100 insulins.
- Do not drop the consumables
- The InsuJet™ should only be used with InsuJet™ Nozzles and Adaptors.
- Check the expiration dates of the InsuJet™ consumables on the outer packaging. Do not use after this date.
- Check the integrity of the InsuJet™ consumables packaging. Do not use if the package is breached.
- Never use the same Nozzle for different types of insulin.
- Never use the same adaptor for different insulin cartridges or vials. Discard the Adaptor with the empty insulin cartridge or vial.
- Be careful not to touch the needle inside the Adaptor.
- Never discharge the InsuJet™ without a Nozzle attached, or air inside the Nozzle. This is called DRY-FIRING and may damage the device beyond repair.
- Replace the Nozzle whenever the InsuJet™ has not been used for more than 3 days.
- Never use the same consumable for different patients.
- Adult supervision is highly recommended when a child uses the InsuJet™. Make sure that children are told that the InsuJet™ is not a toy.
- Keep all medications and injection devices out of the reach of children.
- Never use, or attempt to repair a damaged or broken device.
- The same Nozzle should never be used for more than 56 injections, or longer than 14 days. Replace the

Nozzle every time a new cartridge or vial is used. Using the Nozzle beyond the indicated lifetime will cause insulin leakage and device malfunction, with loss of warranty.

Good hygiene practices

- Wash your hands thoroughly before using the InsuJet™
- Clean the injection site as directed by your Diabetes Care Specialist.
- Avoid unnecessary contact with the Nozzle tip or the areas on the consumables that come into contact with the insulin. Always keep the nozzle cover and adaptor caps on between use. This will prevent possible contamination.

Identification

InsuJet™ Injector


InsuJet™ Nozzle (0.5mL)

P/N: 01075

Nozzle Cap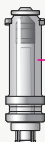

Nozzle

InsuJet™ 3ml Adaptor

P/N: 01055

InsuJet™ 10ml Adaptor

P/N: 01065

3ml insulin cartridge

Cartridge holder

Disposable insulin pen

10ml insulin vial


Example of the use of InsuJet adaptors for 3ml insulin cartridge, disposable penfil or 10ml insulin vials (not included in the InsuJet™ system).


InsuJet™ Nozzle

Replacement and installation


Max **56 injections**, or **14 days** use (whichever comes first).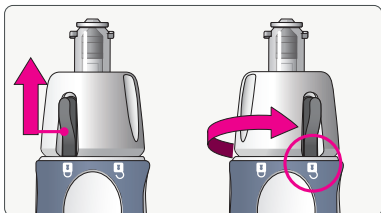


Push the Nozzle Lock Switch upward and Turn the Nozzle Lock until the Switch points to the 'unlocked' position.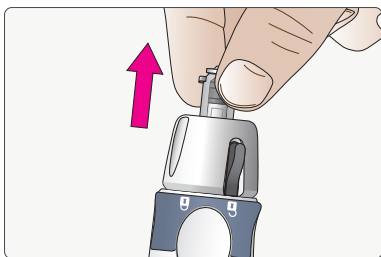


Pull the Nozzle out of the InsuJet™ with a firm straight pull.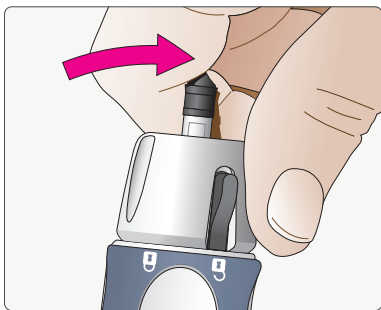


Loosen the internal part of the Nozzle by pushing it firmly to one side.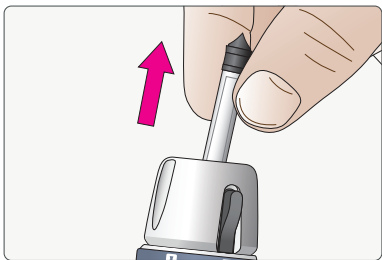

Remove the internal part of the Nozzle and discard it together with the nozzle.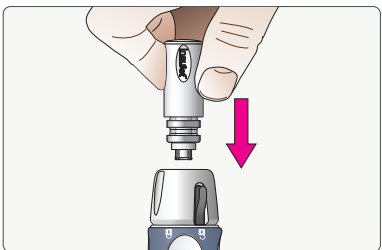

Firmly push the Nozzle into the Nozzle Lock.

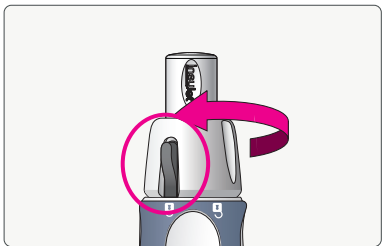Once you have fully inserted the Nozzle, turn the Nozzle Lock until the Switch points to the 'locked' position.

Choose your InsuJet™ adaptor

There are two types of Adaptors available for the InsuJet™.

These Adaptors are used to transport insulin from an insulin cartridge or insulin vial into the Nozzle.


Use the **3ml Adaptor** when using insulin from a **3ml insulin cartridge** or from a **disposable insulin pen**.

Use the **10ml Adaptor** when using insulin from a **10ml vial**.

Removing the consumable from its sterile packaging

Once you confirm you have the correct consumable, confirm the packaging is intact and check the expiration date.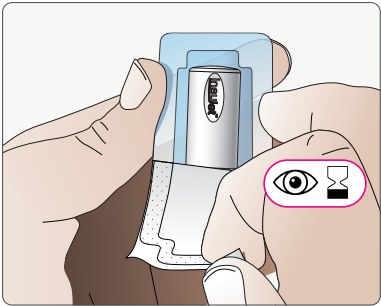


Break or tear a single blister from the strip. Check the expiration date. Open the blister seal by tearing the lid from one of the corners in one clean pull.


Installing the adaptors

InsuJet™ 3ml Adaptor with disposable insulin pen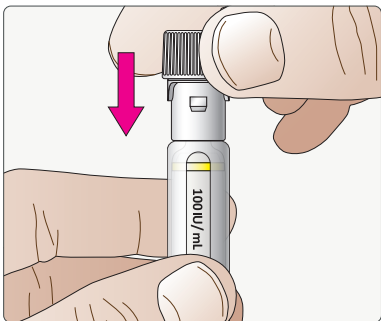

Take the 3ml Adaptor and place it on top of the disposable insulin pen. Push the Adaptor firmly onto the insulin pen until it is fixed.

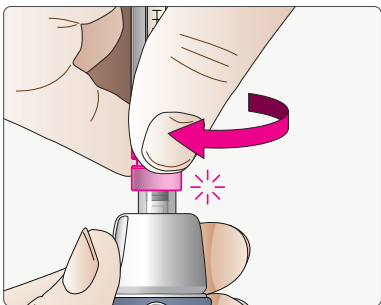
Align the Adaptor with the Nozzle and firmly rotate the Adaptor clockwise onto the Nozzle.

NOTICE:


When attaching the Adaptor with a disposable insulin pen on the Nozzle, make sure to hold the Adaptor when rotating it onto the Nozzle.

InsuJet™ 3ml Adaptor with insulin cartridge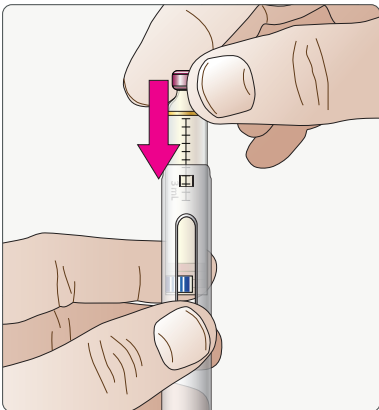

Slide the 3ml insulin cartridge into the Holder, with the cartridge cap facing upwards.

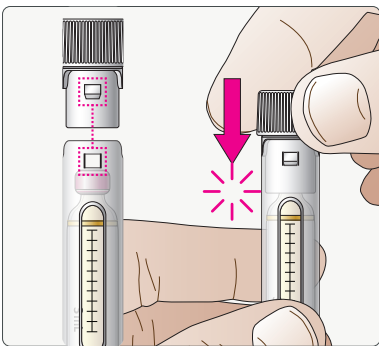
Take the Adaptor and align the protrusions with the corresponding holes in the Holder.

Push firmly until the Adaptor snaps in place. The Adaptor remains on the Holder.

InsuJet™ 10ml Adaptor


with insulin vial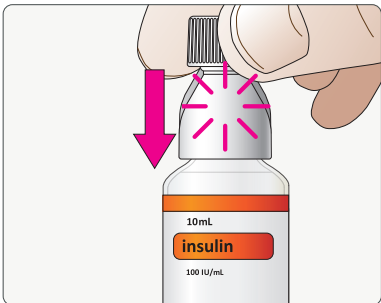

Remove the protective cap from the insulin vial.

Push the 10ml Adaptor firmly onto the vial, until the Adaptor snaps in place. The Adaptor remains on the vial.



Connecting the Adaptor to the Nozzle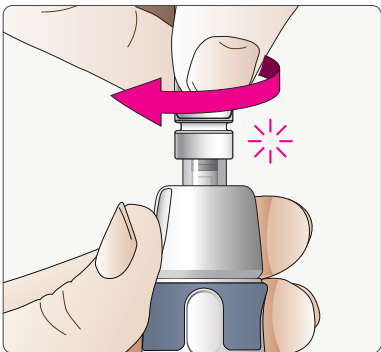


Align the Adaptor with the Nozzle and firmly rotate the Adaptor clockwise onto the Nozzle.

Storage and maintenance

Temperature limits for storage and use: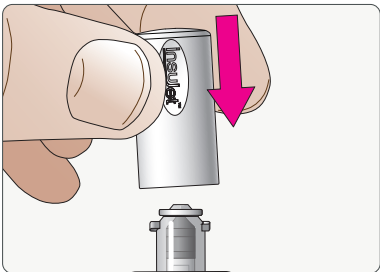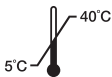

To keep the accessories clean between use, always put back the covers after use.

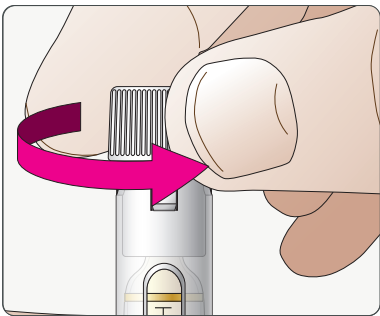Place the Adaptor Cover back on the Adaptor.

Disposal

- All products can be discarded in regular household waste
- Discard the cartridge/vial Adaptor along with each empty insulin cartridge/vial.

Device information

InsuJet Nozzle

- For insulin delivery
- Dose volume: 4-50 international units of U100 insulin per administration, (0.04 - 0.5mL)
- Compatible with InsuJet insulin injector
- Max use: 56 uses or 14 days (whichever comes first).


InsuJet 3ml Adaptor

- InsuJet filling accessory for transfer of insulin
- Compatible with 3ml insulin cartridge and 3ml insulin pen (8mm size).
- Compatible with InsuJet Nozzle.
- One-time installation with cartridge or insulin pen and re-use until the insulin is used up.

InsuJet 10ml Adaptor

- InsuJet filling accessory for transfer of insulin
- Compatible with InsuJet Nozzle
- Compatible with 10ml insulin vials (14mm size)
- One-time installation with vial and re-use until the insulin is used up.

Temperature limits for storage and use:

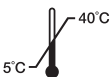Symbol Legend

MEDICAL DEVICE

CONSULT INSTRUCTIONS FOR USE

CAUTION

MANUFACTURER

BATCH CODE

USE-BY DATE

STERILIZED USING ETHYLENE OXIDE IN STERIL BARRIER SYSTEM

TEMPERATURE LIMITS

DO NOT USE IF PACKAGE IS DAMAGED AND CONSULT INSTRUCTIONS FOR USE

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Sol-Millennium Medical, Inc.